

English

BAPA tube

REF	18004	CONTENT	50 tubes
------------	-------	----------------	----------

Intended use

The BAPA tube is intended to be used for the collection, transport, and storage of blood specimens for use with the T-TAS 01 system.

Summary

BAPA (Benzylsulfonyl-D-argininyl-prolyl-4-amidinobenzylamide), an inhibitor of thrombin and factor Xa, allows anticoagulation of blood without interference with physiological calcium levels.

The specified concentration of BAPA in collected blood is $\geq 50 \mu\text{g/mL}$.

Blood collection tube

50 x 3 mL tubes containing spray-dried BAPA anticoagulant

Warnings and precautions

- For in vitro diagnostic use only.
- For use by trained professionals only.
- Human blood should be handled as potentially infectious, and Universal Precautions should be followed.
- Disposal of all waste material should be in accordance with local, state, and Federal guidelines.
- Do not use tubes if foreign matter is present in the tube.
- Do not use tubes after their expiration date.

Precautions for blood collection

- Use 21 gauge or larger-bore needles (18-20 gauge) for blood collection.
- Mix the blood by gently inverting the tube 5 times immediately after filling.

Storage and stability

- Store unused collection tubes at 5 to 30 °C. Avoid exposure to direct sunlight.
- The product is stable until the expiration date printed on the tube label when stored under these conditions.

Note: Deviation from recommended storage conditions may lead to impairment of the tube quality.

References

For additional information about measurements using the BAPA tube, please refer to the T-TAS 01 User Manual and assay package insert(s).

Symbols



: Catalog number




: Batch Code



: Use-by date



: In vitro diagnostic
medical device

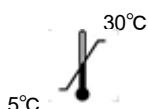

: Temperature limit




: Do not re-use



: Sterile



: Consult
Instructions for
Use

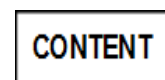

: Quantity per
carton




: CE mark



: Authorized representative
in the European Community



: Manufacturer

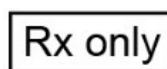

: This product is restricted to sale by or
on the order of a licensed healthcare practitioner



: Importer



ZACROS Corporation

1-1-1 Koishikawa, Bunkyo-ku, Tokyo 112-0002 Japan



Medical Device Safety Service GmbH

Schiffgraben 41 30175 Hannover, Germany



EU Importer

MedEnvoy

Prinses Margrietplantsoen 33 - Suite 123

2595 AM The Hague

The Netherlands

