

## **English**

# BAPA tube

|            |       |                |          |
|------------|-------|----------------|----------|
| <b>REF</b> | 18004 | <b>CONTENT</b> | 50 tubes |
|------------|-------|----------------|----------|

### **Intended use**

The BAPA tube is intended to be used for the collection, transport, and storage of blood specimens for use with the T-TAS system.

### **Summary**

BAPA (Benzylsulfonyl-D-argininyl-prolyl-4-amidinobenzylamide), an inhibitor of thrombin and factor Xa, allows anticoagulation of blood without interference with physiological calcium levels.

The specified concentration of BAPA in collected blood is  $\geq 50 \mu\text{g/mL}$ .

### **Blood collection tube**

50 x 3 mL tubes containing spray-dried BAPA anticoagulant

### **Warnings and precautions**

- For in vitro diagnostic use only.
- For use by trained professionals only.
- Human blood should be handled as potentially infectious, and Universal Precautions should be followed.
- Disposal of all waste material should be in accordance with local, state, and Federal guidelines.
- Do not use tubes if foreign matter is present in the tube.
- Do not use tubes after their expiration date.

### **Precautions for blood collection**

- Use 21 gauge or larger-bore needles (18-20 gauge) for blood collection.
- Mix the blood by gently inverting the tube 5 times immediately after filling.

### **Storage and stability**

- Store unused collection tubes at 5 to 30 °C. Avoid exposure to direct sunlight.
- The product is stable until the expiration date printed on the tube label when stored under these conditions.

Note: Deviation from recommended storage conditions may lead to impairment of the tube quality.

## References

For additional information about measurements using the BAPA tube, please refer to the T-TAS User Manual and assay package insert(s).

## Symbols



: Catalog number




: Batch Code



: Use-by date



: In vitro diagnostic  
medical device

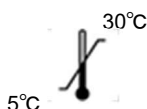

: Temperature limit



: Do not re-use



: Sterile




: Consult  
Instructions for  
Use



: Quantity per  
carton




: CE mark

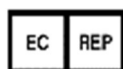

: Authorized representative  
in the European Community



: Manufacturer

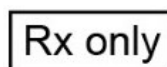

: This product is restricted to sale by or  
on the order of a licensed healthcare practitioner



: Importer



ZACROS Corporation  
1-1-1 Koishikawa, Bunkyo-ku, Tokyo 112-0002 Japan



MDSS GmbH  
Schiffgraben 41 30175 Hannover, Germany



MedEnvoy Global B.V.  
Prinses Margrietplantsoen 33 - Suite 123  
2595 AM The Hague  
The Netherlands

T-TAS® is a registered trademark of ZACROS Corporation.

©2024 ZACROS Corporation

All rights reserve

